

Incisor Speculum



Use of the Swissvet Incisor Speculum:

The Swissvet Incisor Speculum is designed for equine use. It is designed for working on incisor teeth. Use the speculum as shown in picture A.

Make sure the horse is adequately sedated prior to the use of the speculum. Please make sure you understand the possible side effects and danger of incorrect or excessive incisor work in horses.

Do not hesitate to contact Swissvet if you have any questions or concerns.

Bungee cords Warning

- Stretch the elastic cord cautiously, as rebounding hook after uncontrolled release can cause severe injuries to unprotected body parts, particularly to the eyes.
- Adjust length of the elastic cord according to the size of the horses head.
- Do not stretch the elastic cord in the axis of one's face and not beyond 75% of its unstretched length.
- make sure the elastic cord is properly hooked before stretching the cable
- Inspect the bungee cord prior to every use to make sure no alterations, cuts or abrasions are present